



CORFLEX.

Designed for life.

CRYO PNEUMATIC KNEE ORTHOSIS W/ HINGES

1. Remove optional gel pack and place in freezer for a minimum of 2 hours (see warnings below).
2. Remove gel from freezer; re-attach gel pack inside knee orthosis, making sure seams line up above knee cap (patella) opening.
3. Remove stockinette sleeve from plastic bag, grasp end, insert foot into sleeve, and pull up over knee, making sure to center over knee cap (patella).
4. Grasp knee orthosis and locate top (note: valve located at top) and place opening over affected knee cap (patella). **FIGURE 1**
5. Separate hinges and orient so that they flex appropriately, hinges are not right or left and can be placed on either side of the orthosis.
6. Align hinge on the outside of the brace along knee midline, applying to the medial and lateral sides of the knee.
7. Hinges may be contoured, if necessary, for proper fit.
8. When applying hinges, utilize the extension straps included to fully secure hinges to the brace exterior. **FIGURE 2**
9. Secure both compression straps to desired fit. **FIGURE 3**
10. Attach tube with bulb to valve.
11. To inflate, switch to "ON" position, pump bulb to desired compression, making sure not to over inflate. To deflate, press button at base of bulb.
12. When desired compression is attained, switch to "OFF" position and remove tube with bulb from valve connection.



FIGURE 1



FIGURE 2



FIGURE 3

Note: Splint may also be deflated by removing tube while in "ON" position.

WARNING: recommended cold therapy treatment; 20 minutes on, 20 minutes off.

CAUTION:

1. DO NOT OVER INFLATE - improper inflation may inhibit circulation or cause skin irritation
2. Use only after following directions for use
3. Do not use gel pack on area of sensitive skin
4. Cold therapy should not be used by people with circulatory problems unless prescribed by a physician
5. Gel pack is non-toxic – not intended for ingestion
6. Discard gel pack if punctured or torn



TO REORDER

- 72-7270 Cryo Pneumatic Knee Orthosis w/ Hinges, 0 Gel
- 72-7271 Cryo Pneumatic Knee Orthosis w/ Hinges, 1 Gel
- 72-7272 Cryo Pneumatic Knee Orthosis w/ Hinges, 2 Gel
- 72-7275 Hinge Set w/ Extension Straps
- 72-7179 Cryo Pneumatic Knee Gel Pack, EA
- 72-7179-010 Cryo Pneumatic Knee Gel Pack, 10/PK



Suggested Billing Code L-1810