

Lace Align Spinal Orthosis Extension Kit

APPLICATION INSTRUCTIONS

Fitting and Application:

Extension Kit is designed and tested for use only with the Lace Align Spinal Orthoses (sold seperately). Use the Extension Kit when fitting patient's with a waist/hip circumference from 60" to 75".

1. Open the brace and lay flat with the inside surface facing up. Remove grey pull tabs from the belt and place off to the side. Remove side panels (LSO Plus model only) and set aside.




2. Release hook and loop closure at end of belt and slide belt end out through large D-Ring. Angle end of belt for easier removal.



3. With the inside of the extension panel facing up, feed the label end through large D-Ring. Push back the edges of the surrounding fabric and thread through at an angle for easier assembly.



4. Fold over and secure end of belt near the large D-Ring. Overlap opposite end of extension panel with original belt.

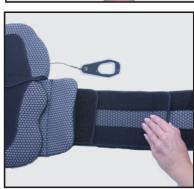

APPLICATION INSTRUCTIONS (cont'd)

5. Adjustments to brace circumference using the extension kit are best made where the panel overlaps the belt. Apply belt and make adjustments at the overlap equally on both sides until proper fit is achieved. Reattach side panels as needed (LSO Plus model only).



Care instructions:

The Lace Align Extension panels may be hand washed using warm water and detergent for delicate fabrics. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer.

Caution:

This product has been designed and tested based on single patient use. Since it is a single use device it should never be used by multiple patients. If any additional pain or symptoms occur while using this device, seek medical attention.

Indications:

Indicated for acute and chronic low back pain, disc herniation, stable, non-displaced fractures and post-op stabilization.

Contraindications:

Contraindicated for unstable, displaced fractures, women who are pregnant, persons with circulatory, pulmonary, cardiovascular or other conditions where increased abdominal pressure could worsen risks.

Warranty:

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex.

To Reorder:

Product Number

Lace Align Extension Kit - 15" 35-3904



CORFLEX, INC. 669 EAST INDUSTRIAL PARK DRIVE MANCHESTER, NH 03109 DESIGNED FOR LIFE.
WWW.CORFLEX.COM
TOLL FREE: 800 426 7353