



CORFLEX®

Designed for life.

CORFLEXGLOBAL.COM | 800 426 7353

CoolTex™ Ag 6" Thumb Spica Application Instruction

Fitting and Application:

1. Release all hook and loop closures. If desired, the removable palmar and dorsal stays and thumb stay may be shaped with a heat gun to fit the curvature of the hand. Remove stays from brace before molding.



2. Place hand into splint, align thumb stay with thumb and put thumb through thumb loop.



3. Close the hook and loop wide elastic strap to desired tension for a comfortable fit.



4. Pull strap tab between index finger and thumb and attach to hook closure on top of splint.



Care Instructions

The CoolTex Ag Thumb Spica may be hand washed using warm water and detergent for delicate fabrics. Remove plastic stays prior to washing. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer. Plastic stays can be cleaned using warm water and a mild disinfectant.

Caution

This product has been designed and tested based on single patient use. Since it is a single use device it should never be used by multiple patients. If any additional pain or symptoms occur while using this device, seek medical attention.

Indications

Indicated for deQuervain's Syndrome, tendinitis, thumb strains or sprains and post-cast removal.

Contraindications

Not for use on patients with circulatory or sensation issues. If swelling, pain, skin irritation or an unusual reaction occurs, discontinue use and consult your medical professional.

Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex Global.

To Reorder

Product Number		Size	Measurement (Wrist Circumference)
Right	Left		
74-5640	74-5650	XS	5"-5 ¾"
74-5641	74-5651	S	5 ¾"-6 ½"
74-5642	74-5652	M	6 ½"-7 ¼"
74-5643	74-5653	L	7 ¼"-8"
74-5644	74-5654	XL	8"-8 ¾"
74-5645	74-5655	2XL	8 ¾"-9 ½"



669 EAST INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109
CORFLEXGLOBAL.COM | 800 426 7353