



**CORFLEX®**

Designed for life.

[CORFLEXGLOBAL.COM](http://CORFLEXGLOBAL.COM) | 800 426 7353

## CoolTex™ Ag 6" Wrist Splint Application Instruction

### Fitting and Application:

1. Release all hook and loop closures. If desired, the removable palmar and dorsal stays may be shaped with a heat gun to fit the curvature of the hand. Remove stays from brace before molding.



2. Slide hand into splint so wide stay fits comfortably in palm.



3. Close the hook and loop wide elastic strap to desired tension for a comfortable fit.



4. Lastly, wrap thumb strap and adjust for a comfortable fit. Strap should fit snug and provide support but not restrict circulation.



## Care Instructions

The CoolTex Ag Wrist Splint may be hand washed using warm water and detergent for delicate fabrics. Remove plastic stays prior to washing. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer. Plastic stays can be cleaned using warm water and a mild disinfectant.

## Caution

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

## Indications

Carpal Tunnel Syndrome, tendonitis, wrist strains or sprains and post-cast removal.

## Contraindications

Do not use on fractures. Not for use on patients with circulatory or sensation issues. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use and consult your medical professional.

## Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex Global.

## To Reorder

Product Number		Size	Measurement (Wrist Circumference)
Right	Left		
74-5620	74-5630	XS	5"-5 ¾"
74-5621	74-5631	S	5 ¾"-6 ½"
74-5622	74-5632	M	6 ½"-7 ¼"
74-5623	74-5633	L	7 ¼"-8"
74-5624	74-5634	XL	8"-8 ¾"
74-5625	74-5635	2XL	8 ¾"-9 ½"



669 EAST INDUSTRIAL PARK DRIVE  
MANCHESTER, NH 03109  
CORFLEXGLOBAL.COM | 800 426 7353