



CORFLEX®

Designed for life.

CORFLEXGLOBAL.COM | 800 426 7353

CoolTex™ Ag 13" Anterior Closure Knee Wrap with Hinge Application Instruction

Fitting and Application:

1. Open knee wrap and unfasten both hook and loop straps. Locate label on inside of brace. Place wrap behind the knee with label located at the top.



2. Center knee wrap on back of leg, position condyle pads at knee joint. Attach calf wrap closure snugly followed by thigh closure.



3. Fasten calf hook and loop strap to desired fit followed by thigh strap.



3. Adjust placement as required. Hinges are removable for washing.



Care Instructions

The CoolTex Ag Knee Anterior Closure Knee Wrap with Hinge may be hand washed using warm water and detergent for delicate fabrics. Remove hinges prior to washing. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer. Hinges are removable.

Caution

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

Indications

Medial/lateral instability, muscle/ligament weakness or injury, degenerative joint disease, post-op rehabilitation, and arthritis.

Contraindications

Not for use on patients with circulatory or sensation issues. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use and consult your medical professional.

Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex Global.

To Reorder

Product Number		Size	Measurement		
Open Popiteal	Closed Popiteal		6" Above Mid-Pat	Mid-Pat	6" Below Mid-Pat
85-1303	85-1313	S	16"-18"	12"-13"	11"-13"
85-1304	85-1314	M	18"-19"	14"-16"	13"-15"
85-1305	85-1315	L	19"-21"	16"-18"	15"-17"
85-1306	85-1316	XL	21"-23"	18"-20"	17"-19"
85-1307	85-1317	2XL	23"-25"	20"-22"	19"-21"
85-1308	85-1318	3XL	25"-27"	22"-24"	21"-22"
85-1309	85-1319	4XL	27"-28"	24"-26"	22"-23"



669 EAST INDUSTRIAL PARK DRIVE
MANCHESTER, NH 03109
CORFLEXGLOBAL.COM | 800 426 7353