



**CORFLEX®**

Designed for life.

[CORFLEXGLOBAL.COM](http://CORFLEXGLOBAL.COM) | 800 426 7353

# CoolTex™ Ag 13" Knee Sleeve with Stays

## Application Instruction

### Fitting and Application:

1. Grasp tabs at wider end of the sleeve, keeping the seam positioned to the back of the leg and the patella opening to the front.



2. Insert foot into sleeve and pull onto leg. Position sleeve so that stays are parallel to each other on the side of the leg and the patella opening is centered on the kneecap.



3. Sleeve should sit on leg with approximately 6" of material above mid patella (kneecap), adjust placement.



## Care Instructions

The CoolTex Ag Knee Sleeve w/Stays may be hand washed using warm water and detergent for delicate fabrics. Remove stays prior to washing. Repeatedly rinse and line dry. Do not use bleach or place in a washer or dryer.

## Caution

This product has been designed and tested based on single patient usage. Since it is a single use device it should never be used by multiple patients. If additional pain or symptoms occur while using this device, seek medical attention.

## Indications

Moderate knee strains or sprains, sports injuries, chondromalacia, post-op rehabilitation and arthritis.

## Contraindications

Not for use on patients with circulatory or sensation issues. If swelling, pain, skin irritation, or an unusual reaction occurs, discontinue use and consult your medical professional.

## Warranty

Products manufactured by Corflex are warranted to the original purchaser against defects in workmanship and material for the life of the product. The foregoing warranty shall not apply to normal wear and tear; or if the product has been subjected to misuse or misapplication the warranty will be voided at the discretion of Corflex Global.

## To Reorder

| Product Number |                 | Size | Measurement      |         |                  |
|----------------|-----------------|------|------------------|---------|------------------|
| Open Popiteal  | Closed Popiteal |      | 6" Above Mid-Pat | Mid-Pat | 6" Below Mid-Pat |
| 85-0152        | 85-0162         | XS   | 16"-18"          | 12"-13" | 11"-12"          |
| 85-0153        | 85-0163         | S    | 17"-19"          | 13"-14" | 12"-13"          |
| 85-0154        | 85-0164         | M    | 18"-20"          | 14"-15" | 13"-14"          |
| 85-0155        | 85-0165         | L    | 19"-21"          | 15"-16" | 14"-15"          |
| 85-0156        | 85-0166         | XL   | 20"-22"          | 16"-18" | 15"-16"          |
| 85-0157        | 85-0167         | 2XL  | 21"-23"          | 18"-20" | 16"-17"          |
| 85-0158        | 85-0168         | 3XL  | 23"-26"          | 20"-22" | 17"-18"          |
| 85-0159        | 85-0169         | 4XL  | 26"-29"          | 22"-24" | 18"-19"          |



669 EAST INDUSTRIAL PARK DRIVE  
MANCHESTER, NH 03109  
CORFLEXGLOBAL.COM | 800 426 7353